# GENUGIB® 3D ARTICULEE

# GENUGIB® 3D HINGED

6904

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EN | Instructions for Use

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## GENUGIB® 3D ARTICULEE

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## GENUGIB® 3D HINGED

Please inspect the device carefully before applying it. If you have any doubts concerning the application and use of this medical device, do not hesitate to consult a relevant healthcare professional.

### INDICATIONS

For knee conditions that may benefit from increased stability around the knee, such as:

- Mild to moderate medial/lateral knee instabilities, sprains and strains.

### PERFORMANCE

The device shall provide increased stability around the knee. The helical straps help reducing the drawer.

### CONTRAINDICTION

Must not be used by individuals for whom compression is contraindicated.

### WARNING • PRECAUTION OF USE

This product contains Natural Rubber Latex which may cause allergic reactions. The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

Users with medical conditions that decrease blood circulation in limbs (e.g. Diabetes or peripheral vascular diseases) should consult their doctor before using the brace.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

When adjusting the strap, tighten it so that it is firm but comfortable.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### FEATURES AND BENEFITS

Intermeshed polycentric hinges and rigid lateral struts.  
Ligament ring.  
X-strap system.

### SET UP AND OPERATION

The first trial should be carried out by a qualified expert to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in exactly the same way.

### The device is supplied with the 2 elastic straps fitted.

#### To set up the device:

**I** Loosen straps 1, 2 and 3 as much as possible.

**II** Put on the orthosis, place the kneecap at the centre of the patellar ring (see schematic A). The GIBAUD logo should be placed at the top.

**III** Tighten the straps in such a way that the hold is firm and comfortable, in tightening order 1,2,3 (see schematic C), i.e.:

1. The rear strap at the bottom.
2. The front strap at the bottom.
3. The front strap at the top.

**IV** For the upper rear part, follow the instructions based on the configuration selected:

#### With the straps 4a et 4b (see schematics B and C):

- Release the round fastening discs from the device.
- Cross the straps 4a and 4b behind the thigh and below the kneecap (see schematic C).
- Secure the round fastening discs in the spaces provided ● while adjusting the tightness in such a way that the hold is firm and comfortable.

#### With the strap 4c (see schematic C):

- Remove the straps 4a and 4b and place the strap 4c in the same way as the straps 1, 2 and 3.
- Tighten the strap in such a way that the hold is firm and comfortable.

## GENUGIB® 3D HINGED

### STORAGE

Store the orthosis away from excessive heat.

### MAINTENANCE

#### BEFORE WASHING:

- Remove all straps, taking care not to tear anything with the fastening tips.
- Remove the frames by the top notch (see schematic D).

#### WASHING

Hand wash the orthosis in warm soapy water (maximum 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat away from excessive heat (e.g. radiators); do not tumble dry; do not iron; do not dry clean.

#### AFTER WASHING:

- Insert the frames as previously removed by the top notches (see schematic D).
- Put the straps back (see schematic B and, if configuration choice with elastic straps, paying attention to the orientation as shown in schematic B) by respecting the strap lengths as indicated (strap 1: medium-sized, strap 2: small, strap 3: large, strap(s) 4: medium-sized or elastic straps).

### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

### MEASUREMENT AND SIZE CHART

Choose the orthosis size in accordance with:  
- Thigh circumference 15 cm from the centre of the kneecap (cm) (A)  
- Knee circumference (cm) (B)

	T1	T2	T3	T4	T5
A	35-45	40-50	45-55	50-60	55-65
B	28-33	33-38	38-43	43-48	48-53

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