G I B A U D

LOMBOGIB® FINESSE

6748/6749

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EN | Instructions for Use

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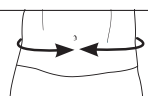
If you have any doubts about the applications and use of this medical device, please ask your doctor and/or fitter, pharmacist, orthotist or orthopaedic consultant for advice.

INDICATIONS

For lower back conditions that may benefit from gross motion restriction, such as:

- Muscular low back pain.
- Sciatica.
- Lumbosacral strains.
- Chronic low back pain.

PERFORMANCE

The device provides lumbar spine gross motion restriction.

The device reduces stress on the low back in flexion and extension.

The device provides pain relief.

CONTRAINDICATION

Must not be used by individuals for whom compression is contraindicated.

WARNINGS • PRECAUTIONS FOR USE

Patients suffering from pelvic floor weakness should consult with their medical professional before using the orthosis.

The orthosis must not be used in direct contact with a damaged skin. Do not wear for extended periods without medical follow-up.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional. The user and/or patient must report any serious incident

that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Gradual withdrawal possibly combined with functional physiotherapy.

FEATURES OF THE DEVICE

The device is an elastic lumbar support belt featuring four conformable dorsal ribs, two ergonomic finger grips to facilitate fitting and a back cushion.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

When fitting the orthosis, the professional should adapt the dorsal ribs to the patient's morphology.

The belt should be worn under clothing or over underwear.

1) Open the belt by unfastening the self-gripping parts from the abdominal breastplate (see diagram **1**).

2) Based on the medical recommendation, reposition the back cushion on the lower part of the self-gripping area (see diagram **2**).

3) Position the lower end of the belt (where the ribs merge) on top of the intergluteal cleft using the finger grip (see diagram **3**). The Gibaud logo must be upright.

4) Centre the dorsal ribs on either side of the spinal column.

5) Close the belt, with the Gibaud logo on the outside (see diagram **4**).

Adapt tightening to the patient's activity and pain intensity to ensure firm yet comfortable fastening (see diagram **5**). Make sure to keep the belt in the position indicated in points 3 and 4.

STORAGE

Store the orthosis away from light, moisture and extreme heat (including radiators and tumble dryers).

MAINTENANCE

Preparation for washing:

Close the belt tightly. Keep the cushion on the belt during washing.

Washing:

Wash gently in warm, soapy water at 30°C (hand or machine wash, delicates cycle); rinse thoroughly; wring out gently; do not bleach; lay flat to dry, do not tumble dry; do not iron; do not dry clean.

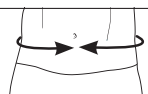
DISPOSAL

The device and packaging must be disposed of in accordance with local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Choose size based on waist circumference (in cm), measured 2cm above the iliac crest.

	0	1	2	3	4	5
	66-80	80-90	90-100	100-110	110-125	125-140

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GIBAUD S.A.S.U