G I B A U D

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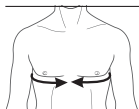
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATION

For arm conditions that may benefit from supporting the arm, such as:

- Post trauma and pre or post-operative.
- To support a cast or a splint.

PERFORMANCE

The devices shall support the arm.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

Do not wear for extended periods without medical follow-up

The orthosis must not be used in direct contact with a damaged skin

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional. The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

- 1 - Slip the forearm into the pouch.
- 2 - Pass the adjustable strap across the back, returning it to the opposite shoulder.
- 3 - Attach the clasp to the buckle on the front of the pouch.
- 4 - Adjust the arm to a 90° angle by attaching the adjustable strap on its hook and loop zone.

CARE INSTRUCTION AND MAINTENANCE

Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations

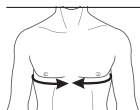
MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the thorax.

For men: measure on the nipples (without the arms) in cm.

For women: measure under the chest (without the arms) in cm.

T00	T0	T1	T2	T3	T4
<45	45-60	60-80	80-100	100-120	120-140

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GIBAUD S.A.S.U