G I B A U D

GENUGIB[®] OPEN

6900

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EN | Instructions for Use

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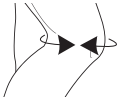
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

INDICATIONS

For knee conditions that may benefit from increased stability around the knee, such as:

- Mild to moderate medial/lateral knee instabilities, sprains and strains.

PERFORMANCE

The device provides increased stability around the knee.

CONTRAINDICATION

None known.

WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

When adjusting the strap, tighten it so that it is firm but comfortable.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Fully opens at the front thanks to two straps (top and bottom).

Two patellar pads (half-rings) enabling to position the patella at the centre.

Dual-centre hinge mechanism and stiff lateral struts increase stability around the knee.

Large opening at the back of the knee.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

Can be worn over or under trousers.

During fitting, the applicator must shape, if necessary, the joints of the orthosis to adapt it to your morphology.

The GIBAUD logo should be on the front at the top of the device when it is correctly fitted.

1 - Loosen the two rear straps and fold the hook and loop part onto the straps.

2 - Detach the two front straps and fold the hook and loop part onto the straps.

3 - Fully open up the orthosis.

4 - Make sure to place the two blue patellar half-rings on either side of the patella and close the hook-and-loop openings / fastenings, starting with the lower one (see illustrative diagram).

5 - The closing order of the straps is indicated by the numbering (from 1 to 4) point (s) in the circles (see illustrative diagram).

6 - Tighten the straps to obtain firm but comfortable maintain.

CARE INSTRUCTION AND MAINTENANCE

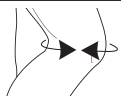
Keep brace away from direct heat sources. After hinges have been removed from the orthosis (see the illustrative diagram with the description of the steps), hand wash the orthosis in warm soapy water (maximum 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, away from direct heat sources (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the knee (in slightly bent position), measured at the middle of the patella (in cm).

	1	2	3	4	5	6
	31-34	34-37	37-40	40-43	43-46	46-50

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GIBAUD S.A.S.U