GIBAUD

**MANUGIB<sup>®</sup> CANAL CARPIEN**

**MANUGIB<sup>®</sup> CARPAL TUNNEL**

6603

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**EN** | Instructions for Use

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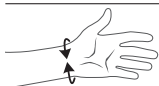
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

## INDICATION

Carpal tunnel syndrome.

## PERFORMANCE

The device shall provide pain relief.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

When adjusting the strap, tighten it so that it is firm but comfortable.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE

Wrist / hand angle, straight reference position with possible settings between +20° and -20°.

Removable and adaptable digital paddle.

Anti-slip pads.

Full opening at the carpal tunnel (no pressure zone).

## SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

1- Completely open the strap under the forearm.

2- Loosen the two other straps above it.

3- Put the fingers (not the thumb) into underside of brace like you would put on a glove **(a)**.

4- Make sure the web space between thumb and index finger rests against the lower edge of the brace **(b)**.

5- The straps only need light tension on them for the brace to be effective.

6- Adjust the wrist joint angle as advised by the doctor. If needed, this angle can be changed to make the brace more comfortable.

NOTE: The metal finger extension bar can be bent. This bar can also be removed (unscrewed); follow the healthcare professional's advice **(c)**.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the orthosis away from sources of excessive heat.

Hand wash in warm soapy water (maximum 40°C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean. The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

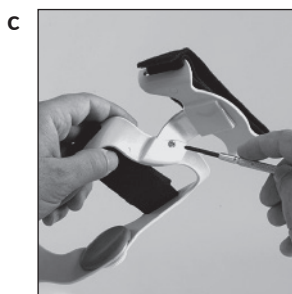
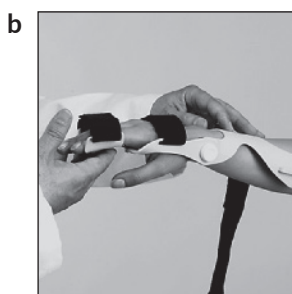
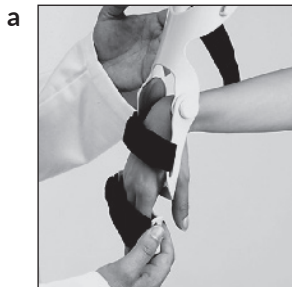
## MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the wrist, below the styloid (in cm).

If between two sizes, choose the upper size.

	1G - 1D	2G - 2D	3G - 3D	4G - 4D
	<14	14-16	17-20	21-24

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