G I B A U D

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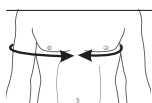
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice

INDICATIONS

For shoulder conditions that may benefit from scapulohumeral immobilization, elbow along the body, such as:

- Shoulder instabilities/injuries.
- Pre or post-operative.
- Soft tissue repairs/strains.

PERFORMANCE

The devices shall provide scapulohumeral immobilization.

CONTRAINDICATION

None known.

WARNINGS / PRECAUTIONS OF USE

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device.

Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SET UP AND OPERATION

The first application should be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

1 - Insert the forearm into the sleeve with the Gibaud mark visible to the person putting it on the patient. Pass the straps over the shoulders and cross them over on the back. **(figure 1)**

2 - Pass the straps under the sleeve and attach them using the special bands. Make sure that the forearm is in a perfectly horizontal position. **(figure 2)**

3 - Attach the chest band to one of the straps leaving the hand and elbow free. Go all the way around the body to attach the band to itself. **(figure 3)**

4 - Tighten the band so that the support provided is firm but comfortable. **(figure 4)**

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

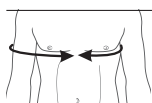
Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently; do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the circumference of the whole «thorax under the chest + humerus (one arm)» in cm.

T0	T1	T2	T3	T4	T5
<45	45-55	55-75	75-95	95-115	115-135

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GIBAUD S.A.S.U