G I B A U D

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**If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.**

## INDICATIONS

Situations requiring inversion/eversion restriction. These may include:

- Acute mild to severe ankle sprains.
- Chronic ankle instability.
- Post-cast and post walker support.
- Ankle injury rehabilitation and/or post-surgical use.

## PERFORMANCE

The device provides inversion and eversion restriction to the ankle.

The device improves proprioception.

## CONTRAINDICATION

None known.

## WARNINGS • PRECAUTIONS OF USE

It's forbidden to drive a vehicle whilst wearing the orthosis.

It is advisable to wear the orthosis with a sock.

When adjusting the straps pay attention to have the tightness firm but comfortable.

Use footwear when walking outdoors or during sportive activities.

Avoid walking with the orthosis without footwear on slippery surfaces, particularly if the surface is wet (bathroom).

The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.

Do not wear for extended periods without medical follow-up.

The orthosis must not be used in direct contact with a damaged skin.

The orthosis is a single patient use device. Do not use for multiple patients.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## FEATURES OF THE DEVICE


Rigid shells.

Resilient foam.


## SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

1- Adjust the separation at the level of the heel in order for the lateral risers to be correctly adjusted on each side of the foot.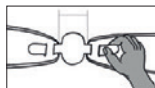


2- Apply the shells on each side of the foot to cover the malleolus. The rivets must be located on the inside part of the leg. Fastening the straps is made from the back towards the front.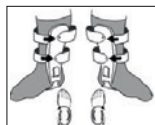

3- Firmly apply the inferior band to have the tightness firm but comfortable.

## MEASUREMENT AND SIZE CHART

Size of the orthosis to choose according to the shoe size (UK, single size).

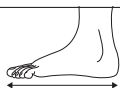1

2,5-11

- 4- Slide the foot wearing the splint into a spacious stable shoe: the footwear must be adjustable by laces or hook and loop system. type attachment. Never use moccasins, ballet shoes or lady's sleepers.
- 5- Adjust the tightening of the footwear.
- 6- Apply the superior band starting from the rear. The tightening set must be firm but comfortable.
- 7- Loosening may occur after some wearing time. In this case retighten the straps.
- 8- Release the straps and re-position the shells.

### CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Keep the device away from sources of excessive heat.

Wipe the plastic parts with a clean moist cloth. The removable side protection can be machine-washed (delicate linens, low temperature program).

Hand wash in warm soapy water (maximum 40 ° C), rinse thoroughly. Wring out gently; do not bleach; dry flat, away from a source of excessive heat (ex: radiators), do not tumble dry; do not iron; do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

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GIBAUD S.A.S.U