G I B A U D

REDRESSE DOS GIBAUD®

GIBAUD® BACK STRAIGHTENER

6611

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EN | Instructions for Use

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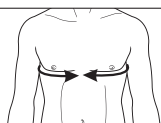
If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopedic consultant for advice.

INDICATIONS

For thoracic conditions that may benefit from posture correction, such as:

- Postural dysfunction
- Back pain

PERFORMANCE

The device provides posture support of CT-spine.

The device improves posture and postural control.

The device provides back pain relief.

CONTRAINDICATIONS

None known.

WARNINGS • PRECAUTIONS OF USE

The orthosis must not be used in direct contact with a damaged skin.

Do not wear for extended periods without medical follow-up.

The orthosis is a single patient use device. Do not use for multiple patients.

Adjust tightening of the straps to obtain a firm but comfortable tightening.

If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

FEATURES OF THE DEVICE

Easy setting up with hook and loop system.

SET UP AND OPERATION

The first application must be carried out by a qualified healthcare professional to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.

1 - Unfold the product.

2 - Open the hook-and-loop system tabs at the end of the straps.

3 - Position the thoracic strap around the rib cage and fasten it by passing the hook-and-loop

through the buckle. The GIBAUD logo located on the back must be positioned the right way up.

4 - Place the supraclavicular straps equipped with a buckle over the shoulders. Then place

the right and left sub clavicular straps under the armpits. Fasten after having passed the hook-and-loop tabs through the buckles.

Adjust tightening of the straps to obtain a firm but comfortable tightening.

CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Hand wash in warm soapy water (maximum 40 ° C), Rinse thoroughly. Wring out gently; Do not bleach; Dry flat, do not tumble dry; Do not iron; Do not dry clean.

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

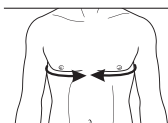
MEASUREMENT AND SIZE CHART

The size is to choose in accordance with the chest circumference (in cm)

Men: measure around the chest on the nipples (without arms).

Women: measure around the chest under the chest (without arms).

0	1	2	3
40-60	60-80	80-100	100-120

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GIBAUD S.A.S.U