G I B A U D

# MALLÉOGIB® 3D

6679

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**EN** | Instructions for Use

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If you have any doubts regarding the applications and use of this medical device, do not hesitate to ask your doctor and / or fitter, pharmacist, orthotist, or orthopaedic consultant for advice.

## INDICATIONS

For Ankle conditions that may benefit from compression, such as;

- Ankle injury rehabilitation.
- Mild sprains and strains.
- Mild instability.

## PERFORMANCE

- The device provides ankle support.
- The device provides proprioception.
- The device provides class 3 compression.

## CONTRAINDICATION

Must not be used by individuals from whom compression is contraindicated.

## WARNING • PRECAUTION OF USE

- The brace should not be used in cases where the injury requires relative or absolute immobilization or surgical procedure.
- Do not wear for extended periods without medical follow-up.
- The orthosis must not be used in direct contact with damaged skin.
- If any problems occur with the use of the orthosis, e.g. pain or appearance of local signs, remove the orthosis and contact your healthcare professional.
- The orthosis is a single patient use device. Do not use for multiple patients.
- The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or

patient is established.

## FEATURES AND BENEFITS

- Latex free.

## SET UP AND OPERATION

- The first trial should be carried out by a qualified expert to understand the correct fitting and how to put the device on properly. Subsequently, the product should be put on the patient in the same way by the user or a family member or a friend.
- The ankle sleeve can be used on the left or right ankle.
- The ankle sleeve is put on by sliding it over the ankle like a sock. The Gibaud logo should be on the top of the ankle.
- Place the lower edge (without logo) under the arch of your foot.

## CARE INSTRUCTION, MAINTENANCE AND DISPOSAL

Hand wash in warm and soapy water (maximum of 40°C); rinse thoroughly; wring out gently;

Do not bleach; dry flat, do not tumble dry; do not iron; do not dry clean.

Store the brace away from sources of extreme heat (including radiators and tumble dryers).

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

## MEASUREMENT AND SIZE CHART

The circumference of your ankle should be measured above the malleoli.

1	2	3	4	5
18-20	20,5-23	23,5-26	26,5-29	29,5-32

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GIBAUD S.A.S.U